



# Clinical Insights

## NAVITOR™ TAVI SYSTEM

# PROCEDURAL AND EARLY OUTCOMES FROM A DUAL-CENTER OBSERVATIONAL STUDY





## Patients

- 39 consecutive patients with attempted TAVI using Navitor.
- Moderate\*, high, or prohibitive surgical risk.
- Severe aortic valve disease (stenosis or regurgitation).
- Navitor was primary option, reserving Evolut to patients with very large annuli.

\*The Navitor™ valve is indicated for transcatheter delivery in patients with symptomatic severe native aortic stenosis who are considered high or extreme surgical risk.

Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**.  
Always check the regulatory status of the device in your region or country.

### PATIENT BASELINE DATA (N=39)

Age	80.0 ± 6.7 years
Women	14 (36.8%)
Prior stroke	3 (7.9%)
Prior pacemaker implantation	4 (16.7%)
Chronic renal failure	4 (16.7%)
Prior aortic valvuloplasty	1 (2.6%)
LVEF	53.4 ± 8.6%
Peak aortic valve gradient	94.3 ± 20.3 mmHg
Mean aortic valve gradient	46.4 ± 15.4 mmHg
TAVI indicated by aortic regurgitation	1 (2.6%)
NYHA class	
I	2 (5.3%)
II	30 (78.9%)
III	6 (15.8%)
IV	0
Logistic EuroSCORE	12.2 ± 8.6%
EuroSCORE II	2.4 ± 1.9
Low surgical risk	2 (5.3%)

Mean ± standard deviation / Numbers (percentage)


# Procedures

PROCEDURAL DATA (n=39)	
General anesthesia	2.6%
Femoral access	94.7%
Axillary access	5.3%
Predilation	71.1%
Postdilation	39.5%
Moderate/severe aortic regurgitation	2.6%
Contrast volume	75.6 ± 14.7 mL
Fluoroscopy time	16.3 ± 2.7 min
Procedural time	61.4 ± 12.3 min
Post-implant ECG changes	21.1%
Post-implant LBBB	18.4%
Post-implant pacemaker dependency	5.7%
Device success	100%
Procedural success	100%
Hospital stay	6.6 ± 4.5 days

Mean ± standard deviation / Percentage

Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**.  
Always check the regulatory status of the device in your region or country.



### Device Size

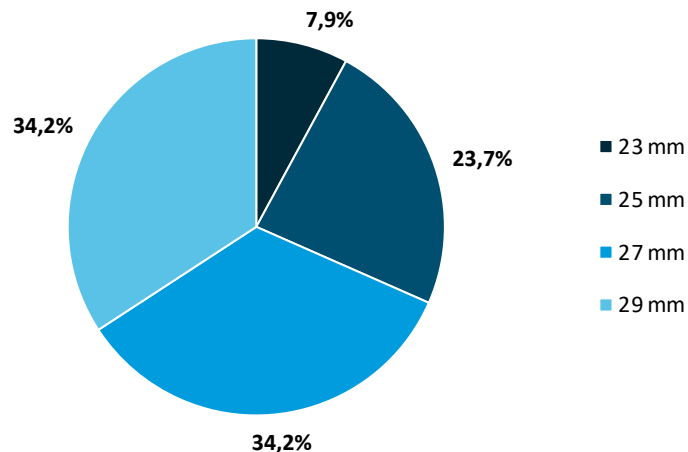

# One-month Outcomes

- No moderate/severe aortic or mitral regurgitation.
- Permanent pacemaker implantation in 1 patient (2.9%).
- No stroke, myocardial infarction, bleeding, access-site complication, rehospitalization occurred up to 1 month of follow-up.
- 1 major adverse event (2.6%): cardiac death\* at 1-month follow-up.

*The favorable impact of the skirt [NaviSeal™ cuff] adjunct seems to be confirmed in this series, with most patients exhibiting no or trace prosthetic aortic valve regurgitation, and no case of moderate or severe regurgitation.*

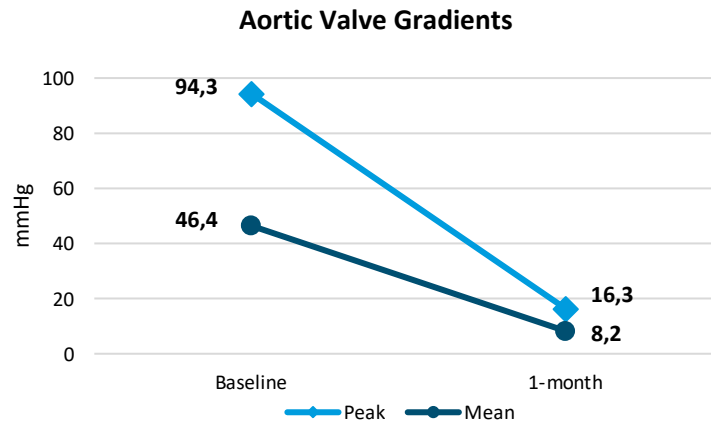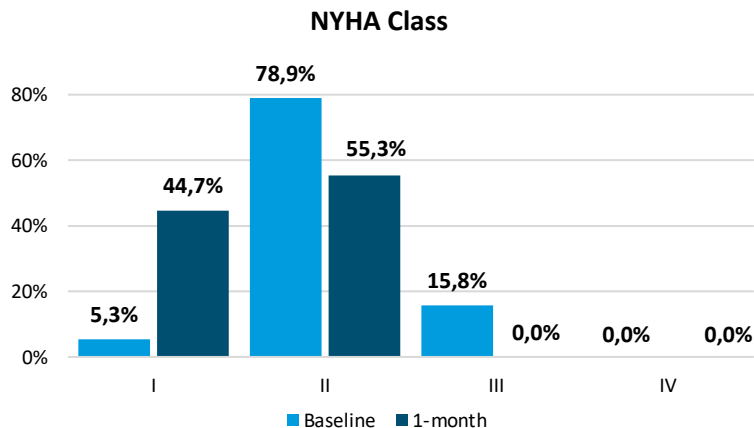

\*Cardiac death in an 87-year-old, frail patient who had been at high surgical risk.

Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**.  
Always check the regulatory status of the device in your region or country.

# Key Takeaway

**NAVITOR IS A USER-FRIENDLY TAVI DEVICE WHICH CAN BE USED IN MOST PATIENTS CONSIDERED ELIGIBLE FOR TAVI**

**VERY HIGH DEVICE AND PROCEDURAL SUCCESS RATES IN EXPERIENCED HANDS**

**FAVORABLE SHORT-TERM CLINICAL OUTCOMES**

- Very low rate of major adverse events
- Low permanent pacemaker implantation rate
- No case of moderate or severe prosthetic aortic valve regurgitation

## CONCLUSION

*The Navitor device, thanks to its unique features, appears a promising technology suitable to further expand indications and risk-benefit profile of TAVI, especially in patients with challenging anatomies, including those at low or moderate surgical risk, given the minimal risk of paravalvular leak.*

**CAUTION:** This product is intended for use by or under the direction of a physician. Prior to use, reference the Instructions for Use, inside the product carton (when available) or at [eifu.abbottvascular.com](http://eifu.abbottvascular.com) or at [medical.abbott/manuals](http://medical.abbott/manuals) for more detailed information on Indications, Contraindications, Warnings, Precautions and Adverse Events.

Information contained herein for **DISTRIBUTION outside of the U.S. ONLY**. Always check the regulatory status for the device in your region.

Illustrations are artist's representations only and should not be considered as engineering drawings or photographs. Photo(s) on file at Abbott.

**Abbott**

3200 Lakeside Dr., Santa Clara, CA 95054 USA

™ Indicates a trademark of the Abbott group of companies.

‡ Indicates a third-party trademark, which is property of its respective owner.

[www.structuralheart.abbott](http://www.structuralheart.abbott)



**Abbott**